


ZIRCOS-E™ ETCHING SOLUTION

“MASTER PLEASE ETCH YOUR CROWN”

30min Instructions

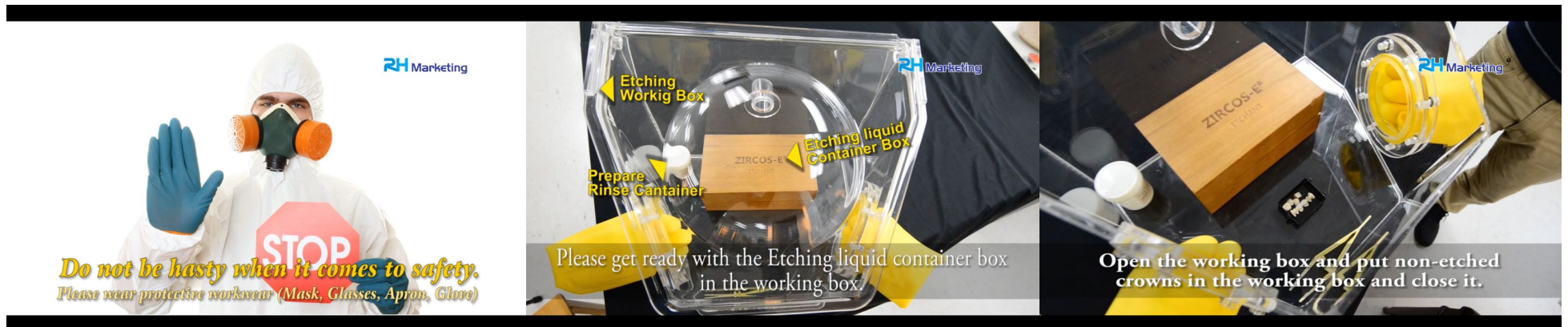




We recommend to use the ultrasonic cleaner in range of 20-60 KHz and 0.2-1.0w/cm2




Please set up the timer exactly 30 minutes and while in this process etching solution will rise the temperature so we recommend to use cold water.



After 30 minutes from Ultrasonic cleaner please wear protective work wear (Mask, Glasses, Apron, and Glove) and open the etching solution container very slowly and cautiously.

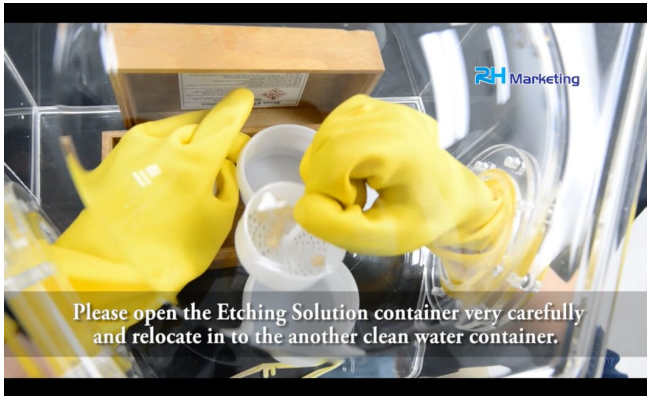

Please open the Etching Solution container very carefully and relocate in to the another clean water container.

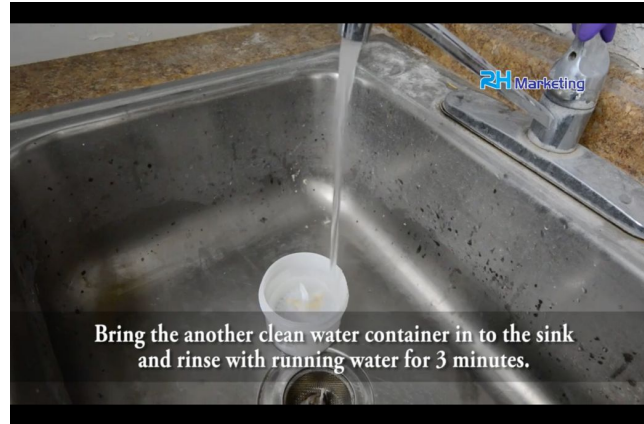

Bring the another clean water container in to the sink and rinse with running water for 3 minutes.



Please make sure your left over Etching Solution container is closed tight for next time use.



Please clean the etched crowns with steamer.



Finally, please deglaze the etched crowns using porcelain furnace with glaze program.

ZIRCOS-E Etching Solution
(Contains Hydrofluoric Acid (HF), Hydrogen Chloride (HCl), Sulfuric Acid (H2SO4), Nitric Acid (HNO3), Phosphoric Acid (H3PO4))

May intensify fire. May be fatal if swallowed and enters airways.
Causes severe skin burns and eye damages. Fatal if inhaled.
Causes damages to organs through prolonged or repeated exposure cause the hazard.
May cause genetic defects. Very toxic to aquatic life.

Keep away from heat, spark, open flames, or hot surfaces - No smoking. Avoid breathing dust, fume, gas, mist, vapors, or spray.
Wash the hands thoroughly after handling. Do not eat, drink or smoke when using this product.
Use only outdoors or in a well-ventilated area. Wear protective gloves, protective clothing, eye protection, and face protection.
Wear respiratory protection.

First aid: IF SWALLOWED: Immediately call a POISON CENTER or doctor/physician. Rinse mouth. Do NOT induce vomiting.
IF ON SKIN (or hair): Remove or take off immediately all contaminated clothing. Rinse skin with water or shower.
IF INHALED: Remove victim to fresh air and keep at rest in a position comfortable for breathing.

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.

Fire: Fire Hazard, Pressure Hazard, Reactivity Hazard, Immediate Hazard
Distributor: RH Marketing, Inc. 18213-A Flower Hill Ways Gaithersburg, MD 20879. Tel: (240) 631-6700