





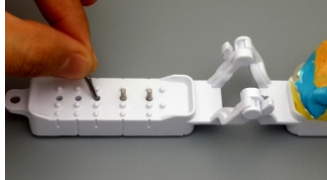
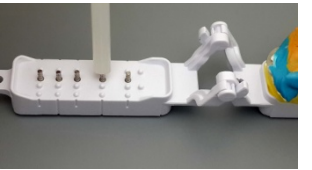






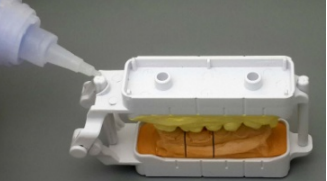



# RH Marketing

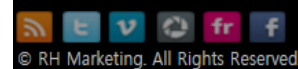
## Smart Base® Active Quadrant Tray Technique

|   |  |   |   |
|---|--|---|---|
|                |   |                         |                    |
| <p>1. Get ready with one set of Smart Base Active. (position pin, die stone pin, guide pin)</p> | <p>2, Trim the useless impression material with scissor.</p>   | <p>3. Make sure not to trim the margin of the impression.</p>   | <p>4. Pour the opposing side first. This will help stabilize the impression position on the tray.</p> |
|                |   |                         |                    |
| <p>5. Make sure the prep teeth are aligned with the row of pin holes.</p>                       | <p>6. Check with the position pin tool to find exact location.</p>   | <p>7. Insert the pin into the pin holes.</p>  | <p>8. Use any tool to push the pin more into the Smart Base Active.</p>                               |
|               |   |                       |                  |
| <p>9. Pour the die stone onto the working tray.</p>   | <p>10. Bend over the Smart Base Active. Make sure to use enough stone so that the trays are parallel and are not closed too tightly.</p> | <p>11. Reduce grinding time by cleaning the excess stone with a small spatula. And let the stone set.</p> | <p>12. Remove the impression.</p>   |
|               |   |                       |                  |
| <p>13. Trim the die stone model.</p>  | <p>14. Make sure to use the mark on the bottom of Smart Base Active when sawing the die stone model.</p>                                 | <p>15. Glue the guide pin to hold the model firmly.</p>   | <p><b>Final.</b></p>  |

Contact us  
18213-A Flower Hill way  
Gaithersburg, MD 20879

Email: [info@rhsupplies.com](mailto:info@rhsupplies.com)  
Website: [Rhsupplies.com](http://Rhsupplies.com)  
Phone: 1-240-631-6700

Follow us



© RH Marketing. All Rights Reserved.